




简易便厕床  
安全使用与组装

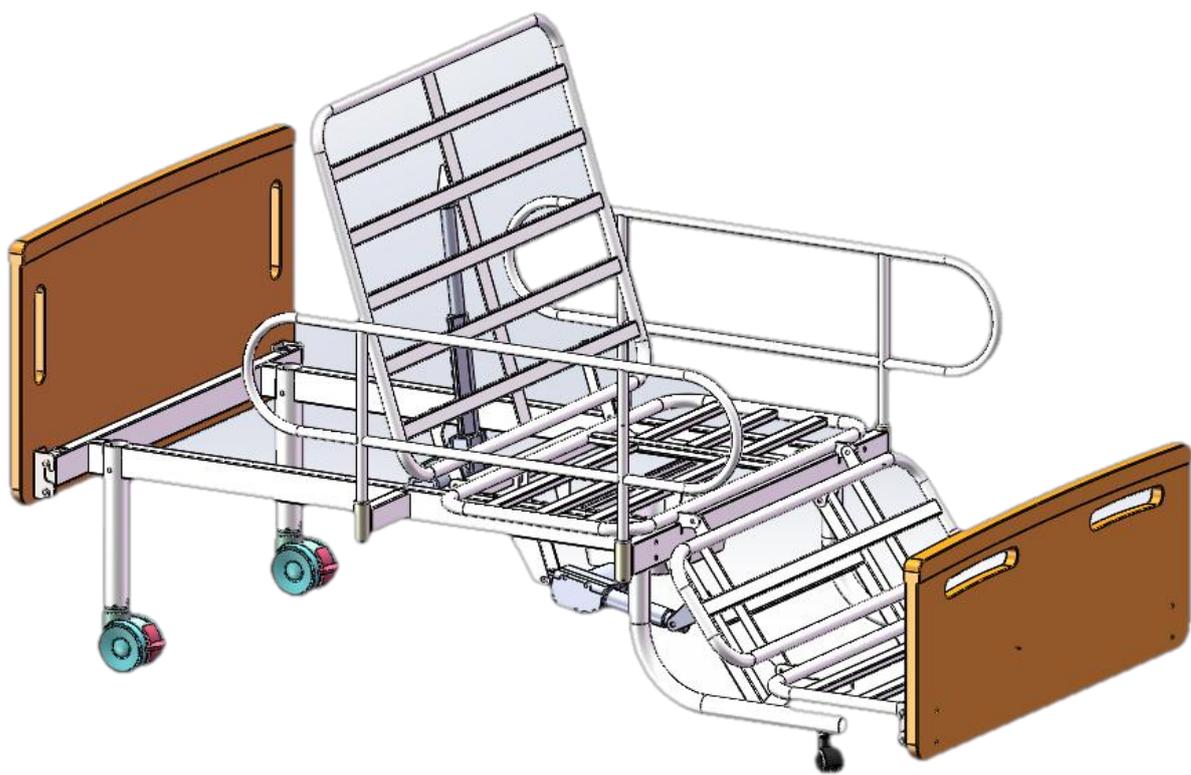


## 01 所有安装配件

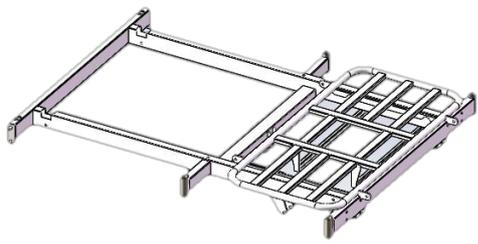

1

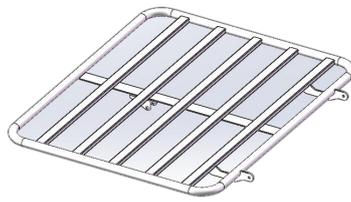


2

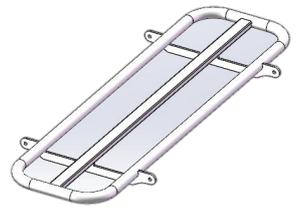


3



4

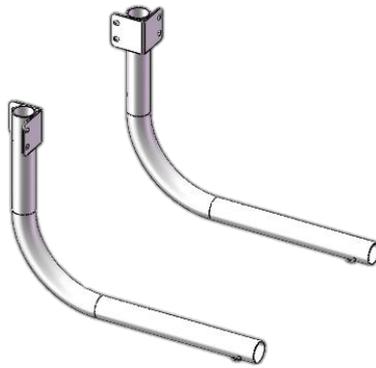


5

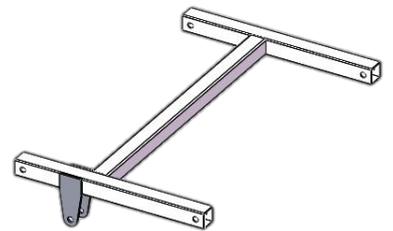

6



7

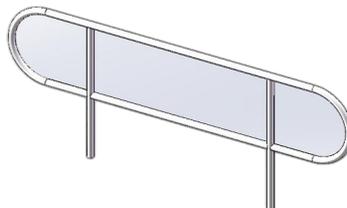


8



9




10

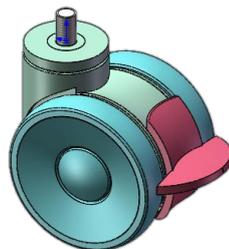

11



12

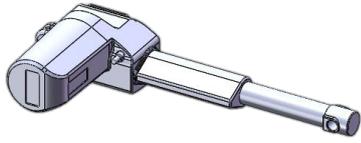

13




14


序号	名称	规格	数量
1	床架		1
2	背板面		1
3	大腿面		1
4	小腿面		1
5	床尾床腿		1 副
6	腿部升降转轴		1
7	床头立腿		2
8	护栏		2
9	床头		1
10	床尾		1
11	床头部位脚轮		2
12	床位部位脚轮		2
13	腿部电机	KDYJT014-081	1
14	背部电机	KDYJT014-080	1

	便盆		1
	半圆头内六角	M8*30	6
		M8*35	8
		M8*70	2
		M8*60	2
		M8*45	8
		M6*10	4
		M10*50	2
	塑料平垫圈	Φ 8*5	6
	自锁母	M8	26
		M10	2
	铁平垫圈	Φ 8	26
		Φ 10	8
	圆柱销	Φ 10*50	6
	不锈钢卡扣	Φ 10	2
	B 型卡		4
	内塞	Φ 38	4
		Φ 50	2
	弹垫	Φ 12	4


## 02 框架的安装步骤

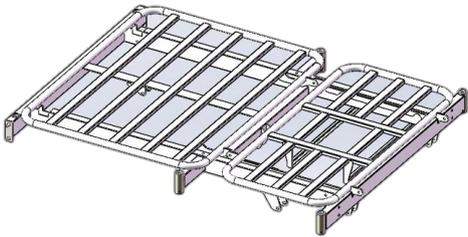


1. 取出带背板的床架

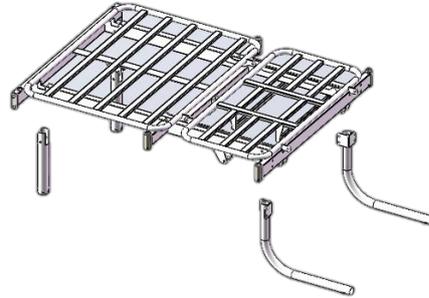


2. 取出 4 个床腿并对齐相关位置

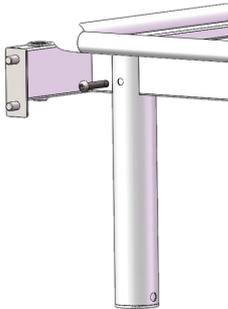


3. 取 M8\*60 内六角螺栓并对齐孔

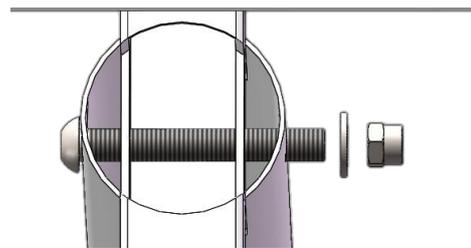


4. 加  $\Phi 8$  铁平垫并用扳手拧紧螺母

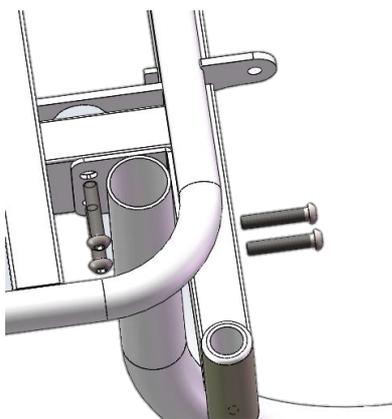



5. 取 M8\*45 内六角螺栓并对齐孔

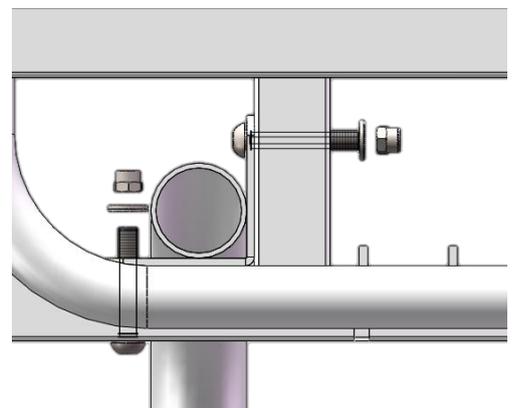




6. 加  $\Phi 8$  铁平垫并用扳手拧紧螺母

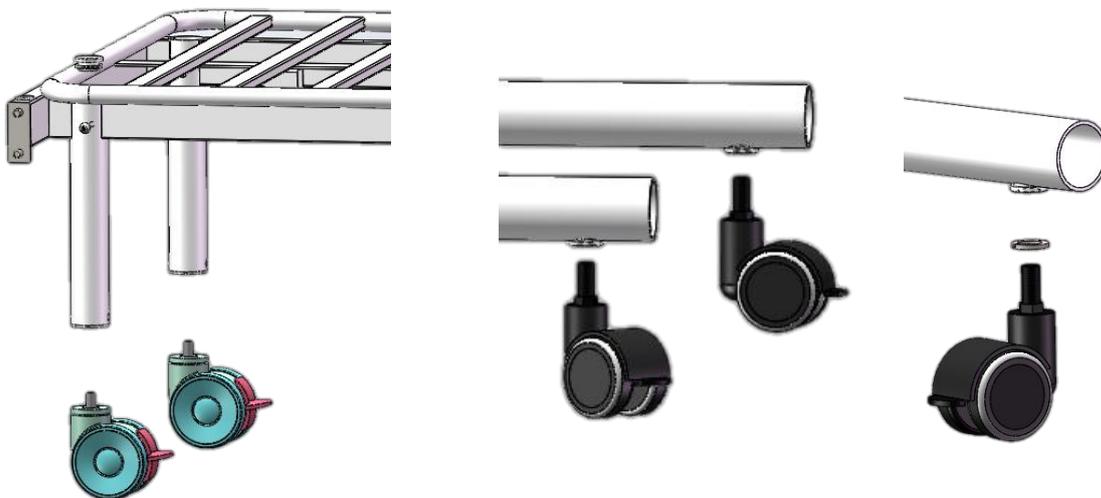


7. 取四只脚轮并按图对齐相关位置并拧紧，床尾脚轮须加  $\Phi 12$  弹垫

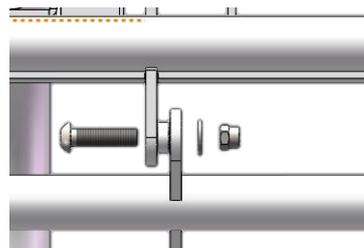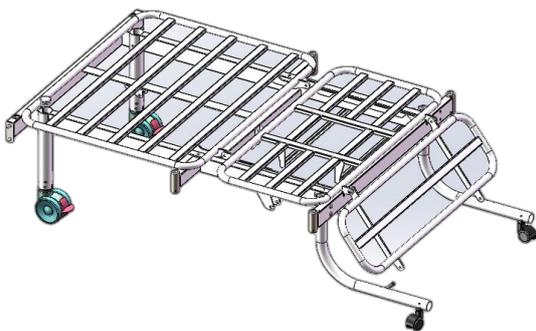


8. 取大腿板按图对齐孔，并取得  $M8*30$  内六角螺栓、 $\Phi 8*5$  塑料垫圈、 $M8$  螺母按图对齐并用扳手拧紧两侧相同。小腿板采取同样方式安装

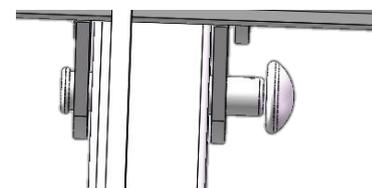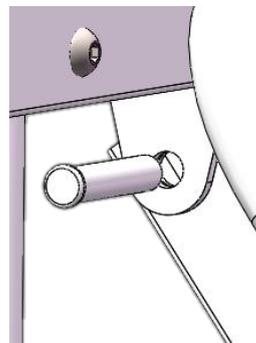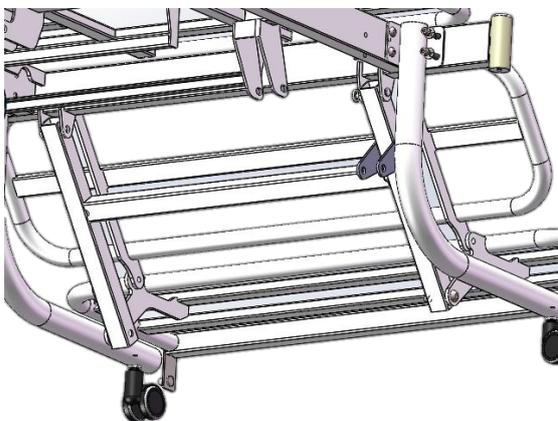


9. 取腿部升降转轴并按图对齐相关孔(注意电机连接座的方向), 取  $\Phi 10*50$  圆柱销穿入与床架连接孔并用  $\Phi 10$  卡扣扣紧两侧相同。

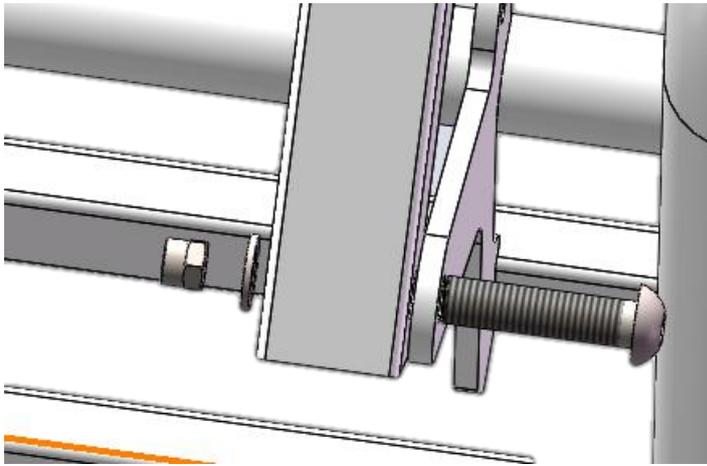


取 M10\*50 半圆头内六角穿入与小腿板连接孔，加  $\Phi 10$  铁垫圈并用扳手拧紧螺母，两侧相同。

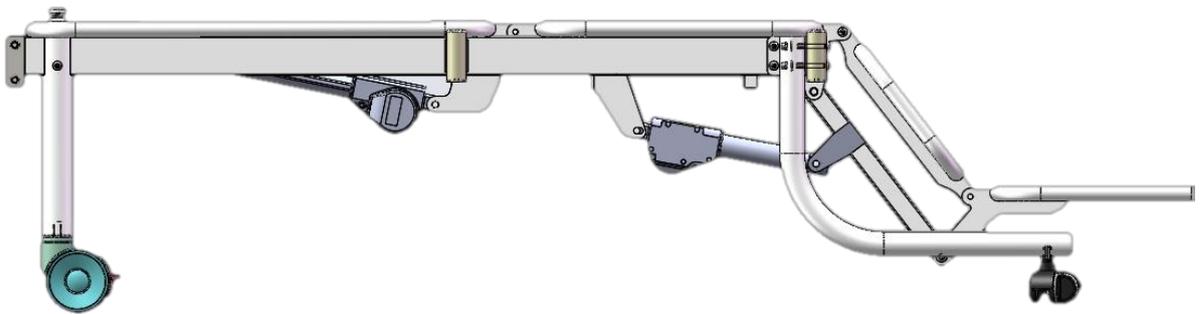

10. 取两个电机按图指示方向对齐孔，取  $\Phi 10*50$  圆柱销穿入并用 B 型卡扣固定圆柱销，并把电机插头插入控制盒对应插孔。

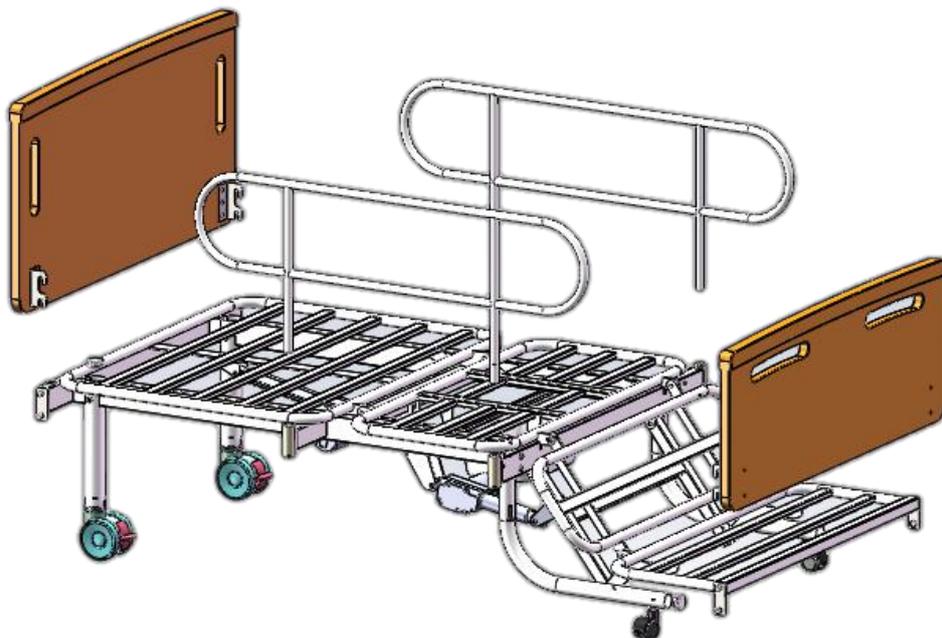

11. 取床头护栏按图对齐，并向下插入固定。

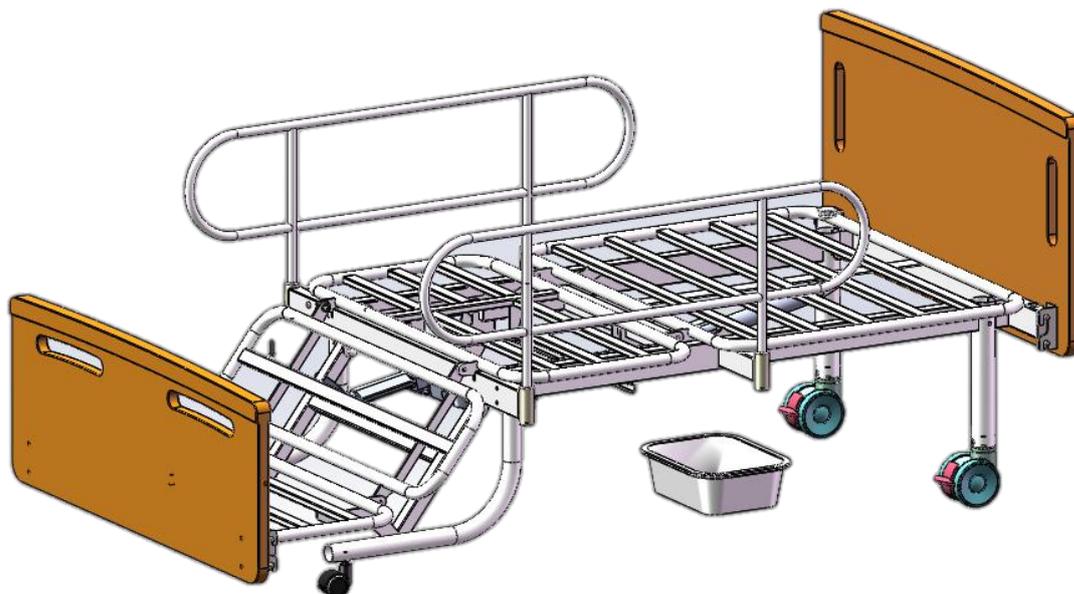

12. 取便盆按图对齐滑槽，并向内推入对齐便孔完成安装。



13. 使用无线手控器操作即可，H 代表背板操作，F 代表腿部操作